Vaginale dilatatieset

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Vaginal Dilation Set

Consisting of 4 dilators of differential length and diameter, with a grip, in a durable cassette.

Before use:

Upon receipt, inspect the set for missing or damaged items and contact your supplier to report any problems or when the product does not meet your expectations. Do not use if damaged. An intact tamper-proof packet shows that it the dilation set is clean and untouched. The dilation set is supplied non-sterile. Before using the product for the first time, we would advise cleaning the dilators with soap and subsequently rinsing them thoroughly.

Directions for use:

This dilation set is strictly for single-patient use. After use, the parts, dilators and grip, have to be washed in warm soapy water and be rinsed well, removing all traces of soap. Dry completely before placing in the storage cassette. The enclosed lubricant has been dermatologically tested and quality-approved by TÜV.

See your healthcare provider for individual instructions.

General instructions for using vaginal dilators:

1. Attach the dilator you want to practice with to the grip. Find a private place and lie comfortably on your back, supporting head and shoulders, with your legs bent and spread apart (or stand with one leg raised onto an elevation). Spread some lubricant around the dilator tip as well as the opening of your vagina.
2. Stress and relax the muscles of your vagina a few times. Gently insert the dilator during a relaxation moment into your vagina for as far as it is comfortable. You can feel some pressure but it is supposed not to hurt. Depending on your disease (condition), you can either move the dilator or keep it in its place for some time.
3. If required (or if you so desire) you can repeat the procedure with a dilator of a larger diameter.

Frequency and length of the treatment will depend on the nature of your disease (condition) and are best discussed with your healthcare provider.

There are no absolute contra-indications known. When pregnant, please first contact your physician/general practitioner or obstetrician/midwife before using the dilation set.

Important notice: This information is not meant to replace medical treatment or advice. Spotting/light bleeding or discomfort may be experienced when using vaginal dilators. If complaints of this kind persist or become worse, discontinue using the dilators and consult a physician.

Notice: Do not autoclave. Do not clean in water hotter than 95°C. Examine the dilators and the grip for damages before each use. When damaged, the vaginal dilation set must not be used.

Disposal: The Vaginal Dilation Set contains materials that can be harmful for the environment. Before disposing of these materials, enquire about the local waste management regulations.

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